Upgrade · The Home Creator Way

LIFT RECLINER CHAIR

**MODEL: SX-81726-US-G/SX-81726-US-B
SX-81726-EU-G/SX-81726-EU-B**

MODEL: SX-81726-US-G/SX-81726-US-B
SX-81726-EU-G/SX-81726-EU-B

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Assembly precautions

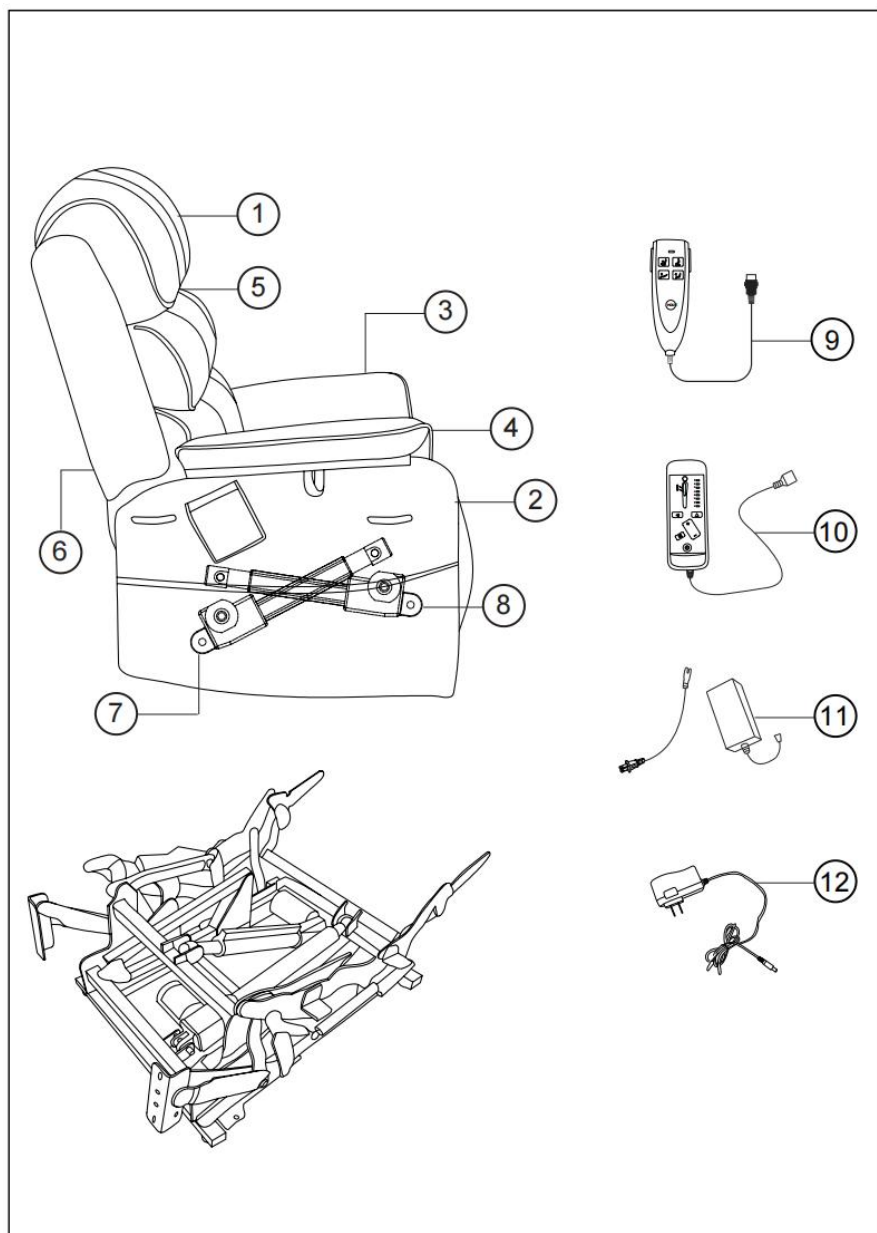
1. Assemble only according to these instructions. Improper assembly may create hazards.
2. Wear ANSI-approved safety goggles and heavy-duty work gloves during assembly.
3. Keep the assembly area clean and well-lit.
4. Keep bystanders out of the area during assembly.
5. Do not assemble if tired or when under the influence of alcohol, drugs or medication.
6. The product capabilities apply to properly and completely assembled products only.
7. Assemble on a flat, level, hard and smooth surface capable of safely supporting the Lift Recliner Chair.
8. For additional information regarding the parts listed in the following pages, please refer to the Assembly Diagram of this manual. Unwrap and separate all parts in a clean work area.

Use precautions

1. Use as intended only.
2. Do not exceed weight capacities. Empty Lift Recliner Chair before moving.
3. Inspect before every use; do not use if parts are loose or damaged.

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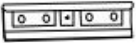
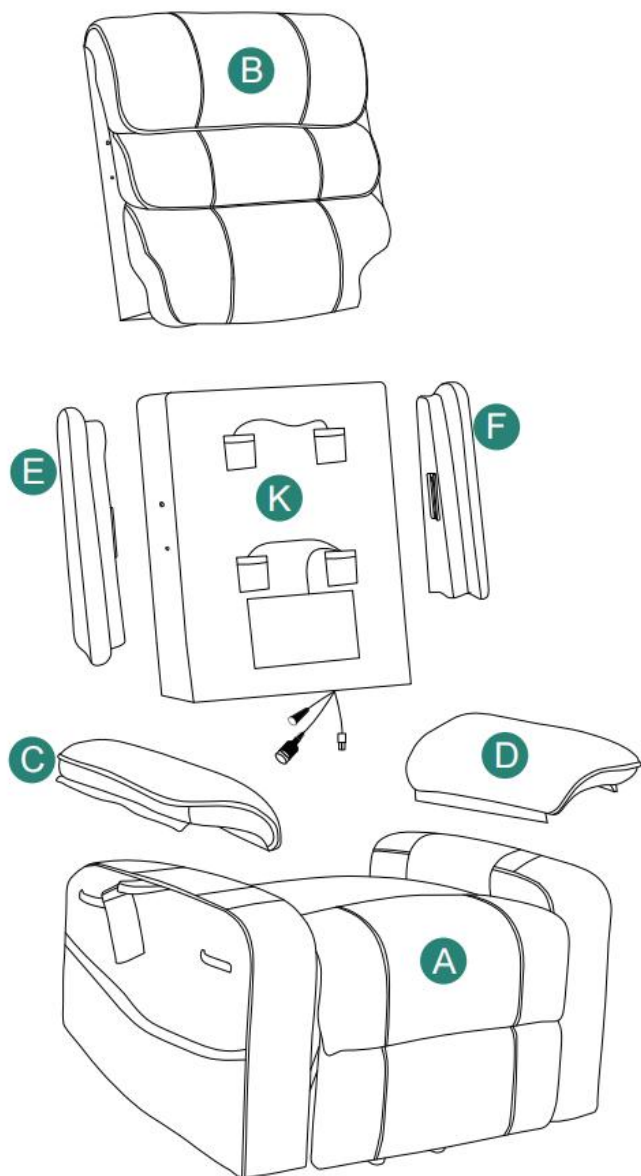
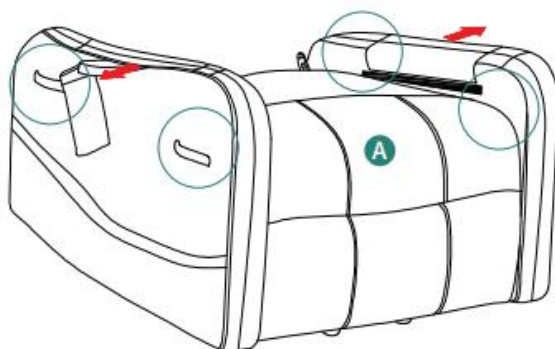
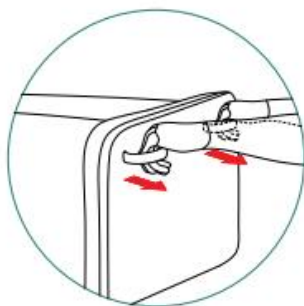
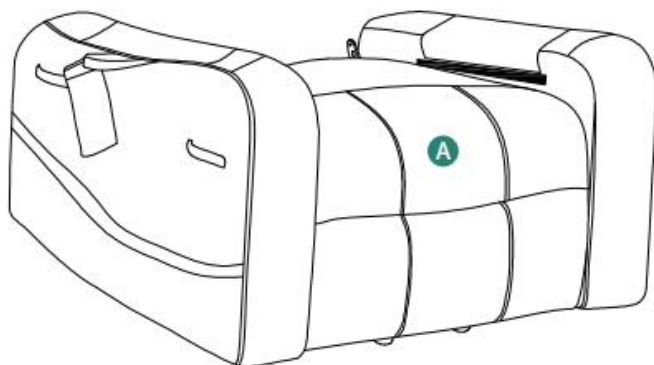
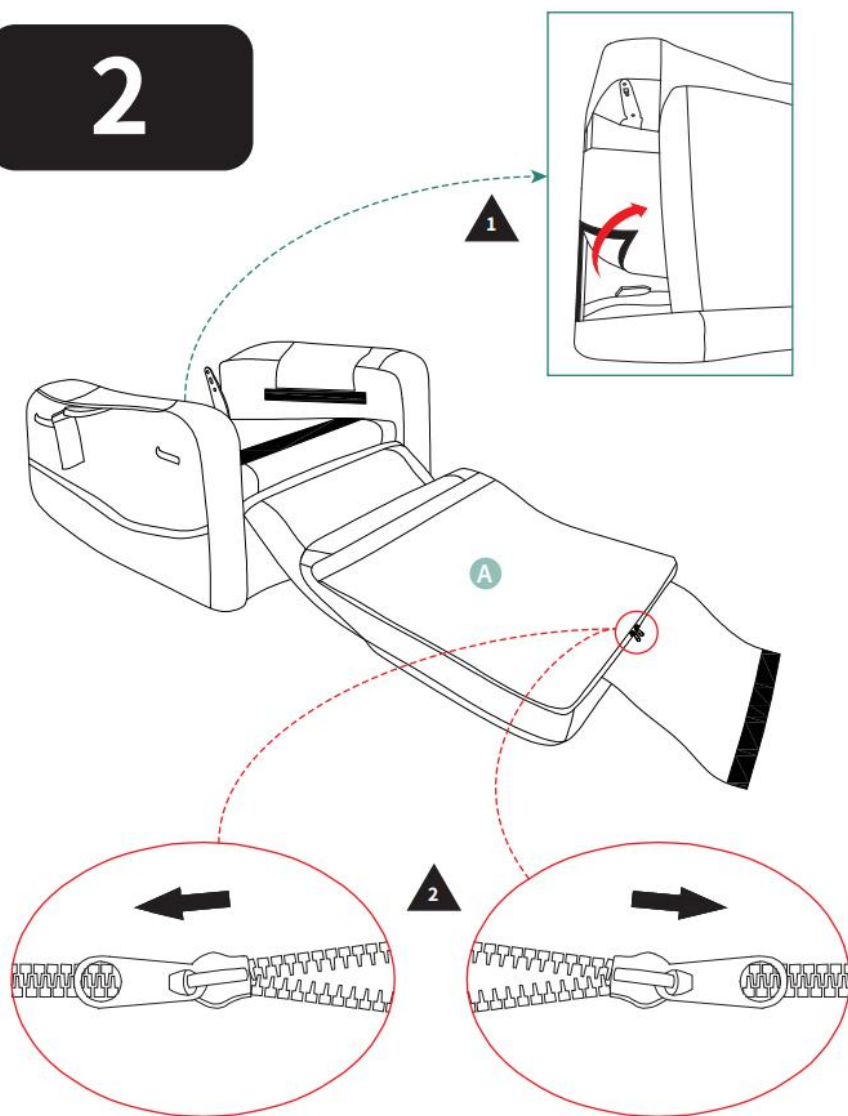
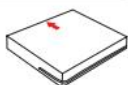
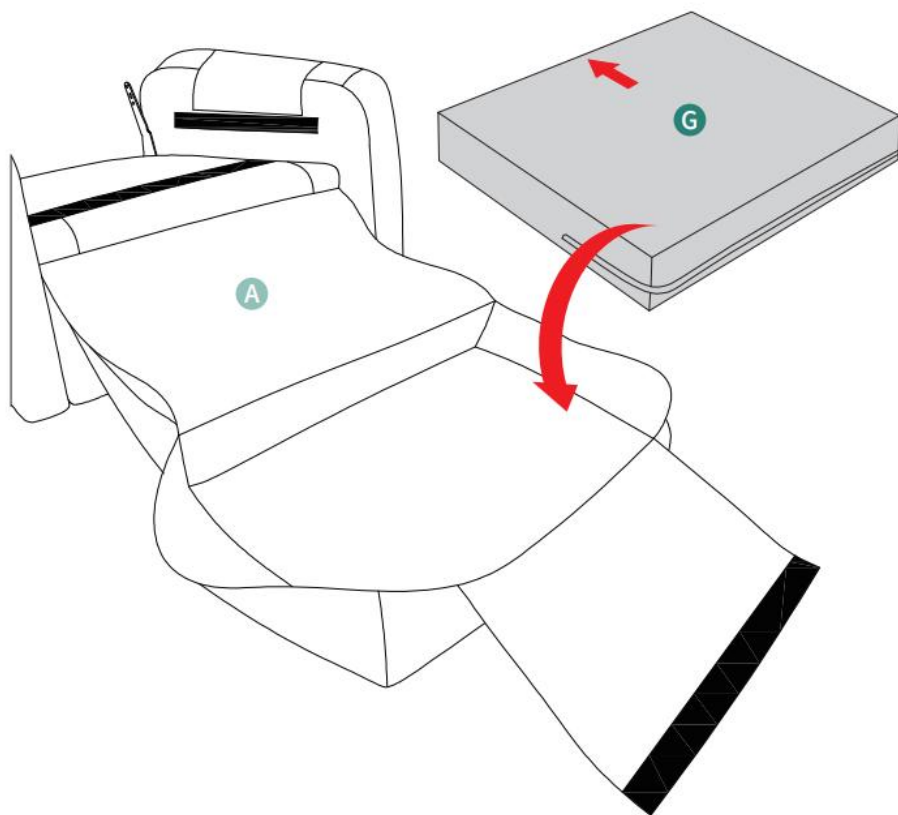
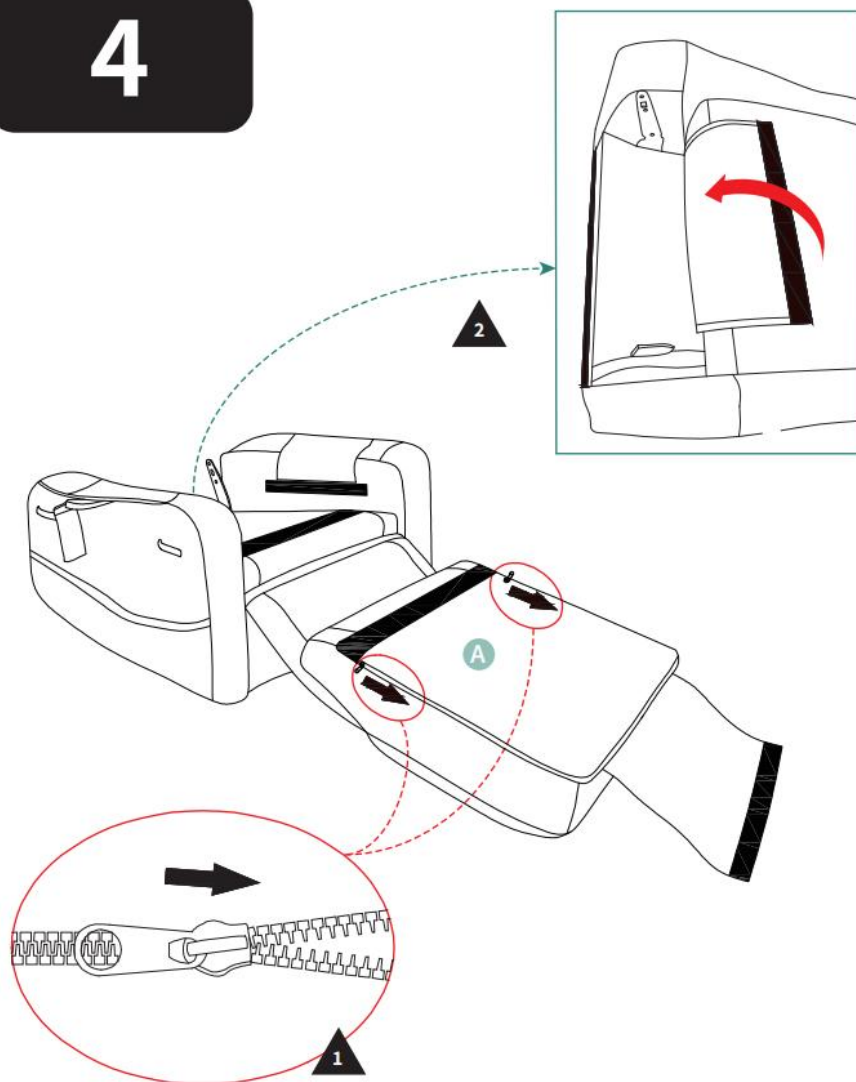
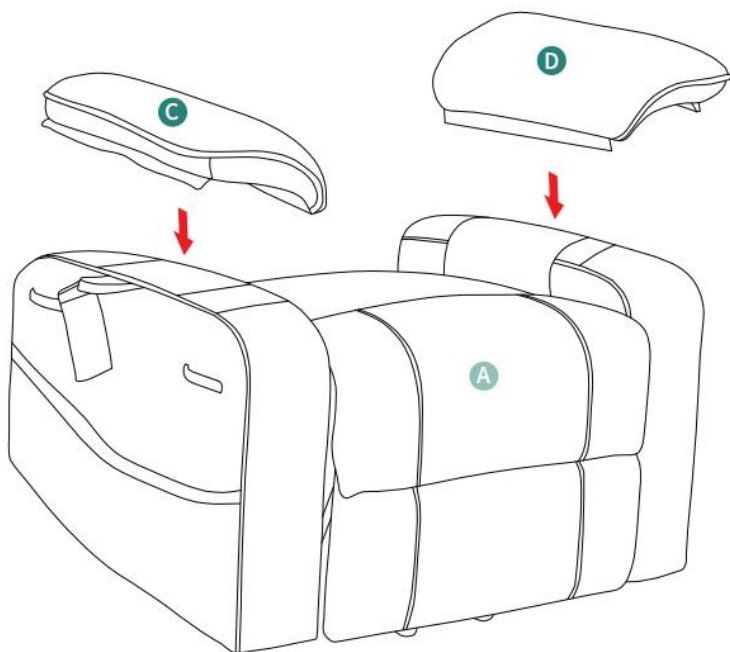
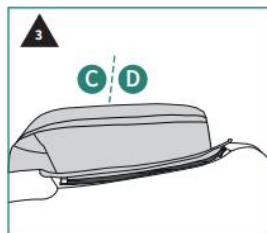
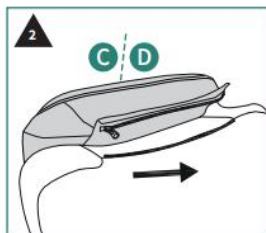
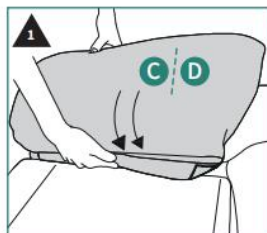
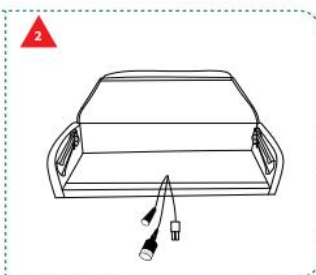
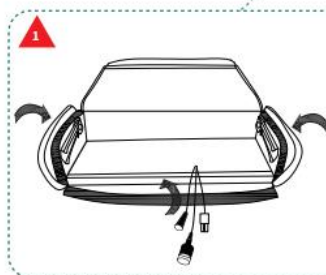
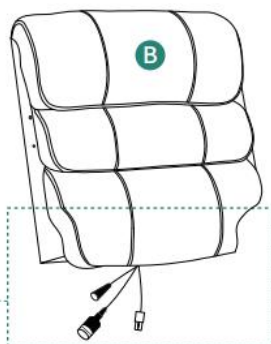
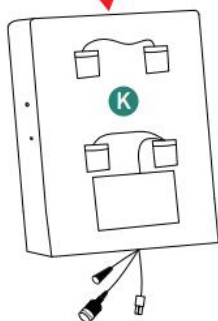
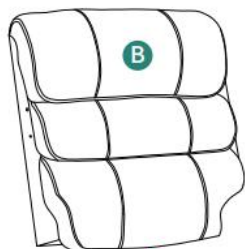
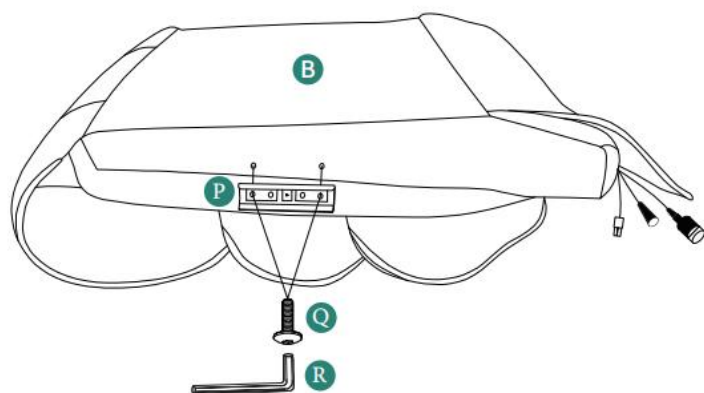
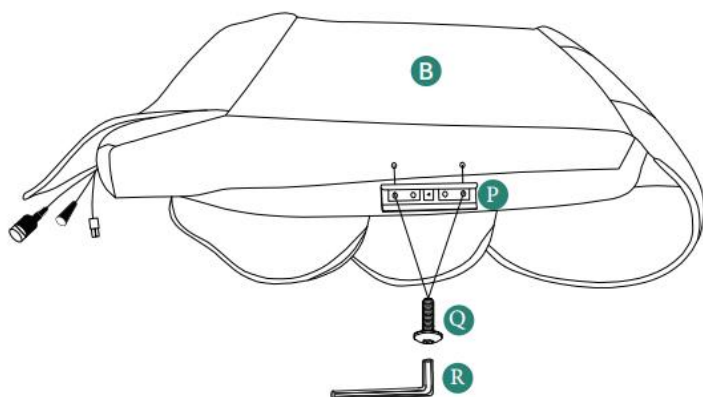
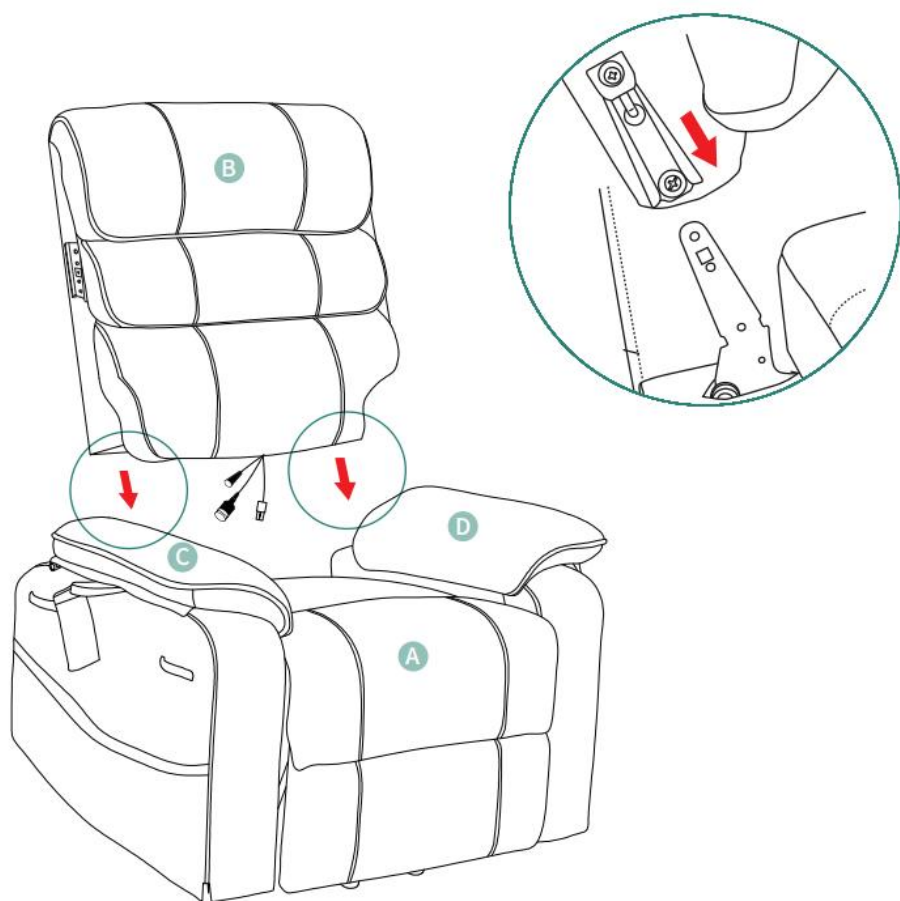
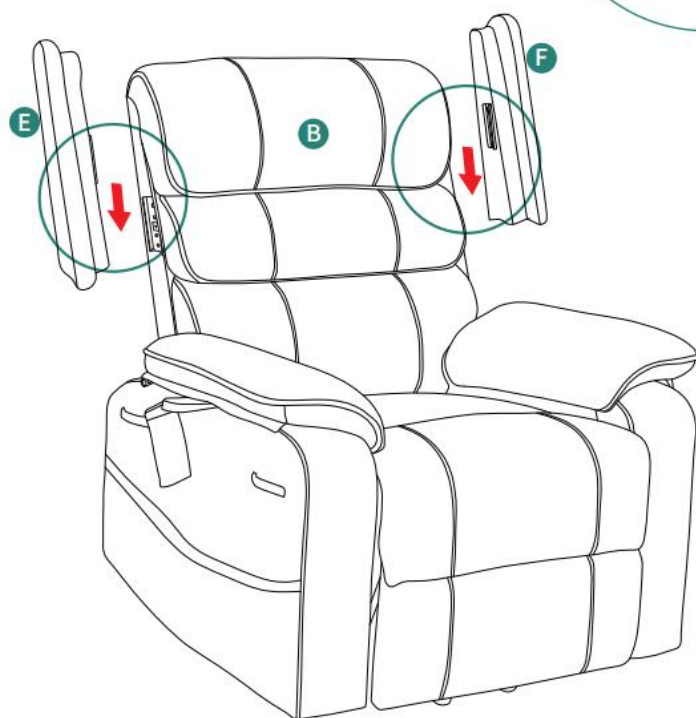
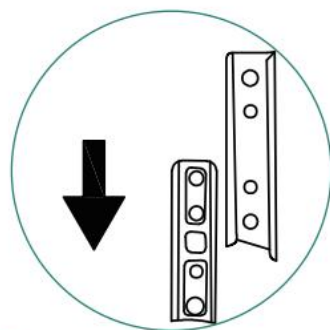
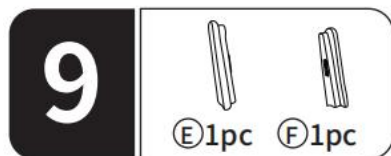
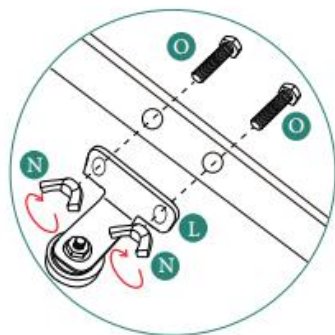
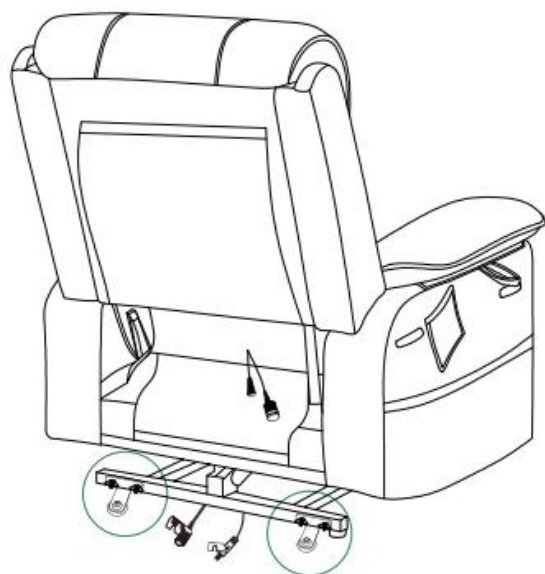
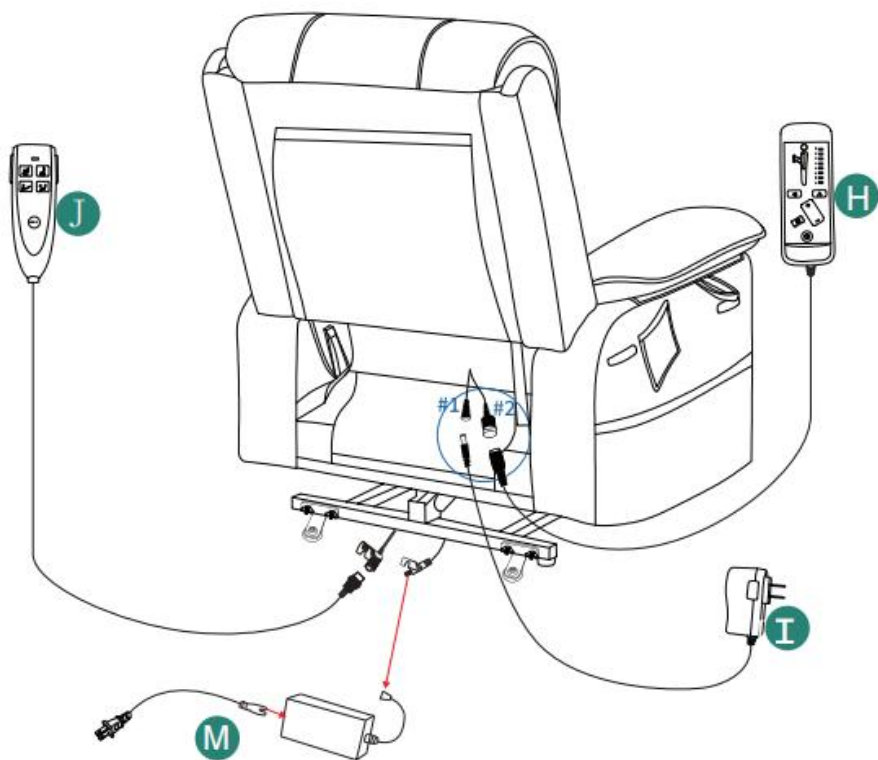
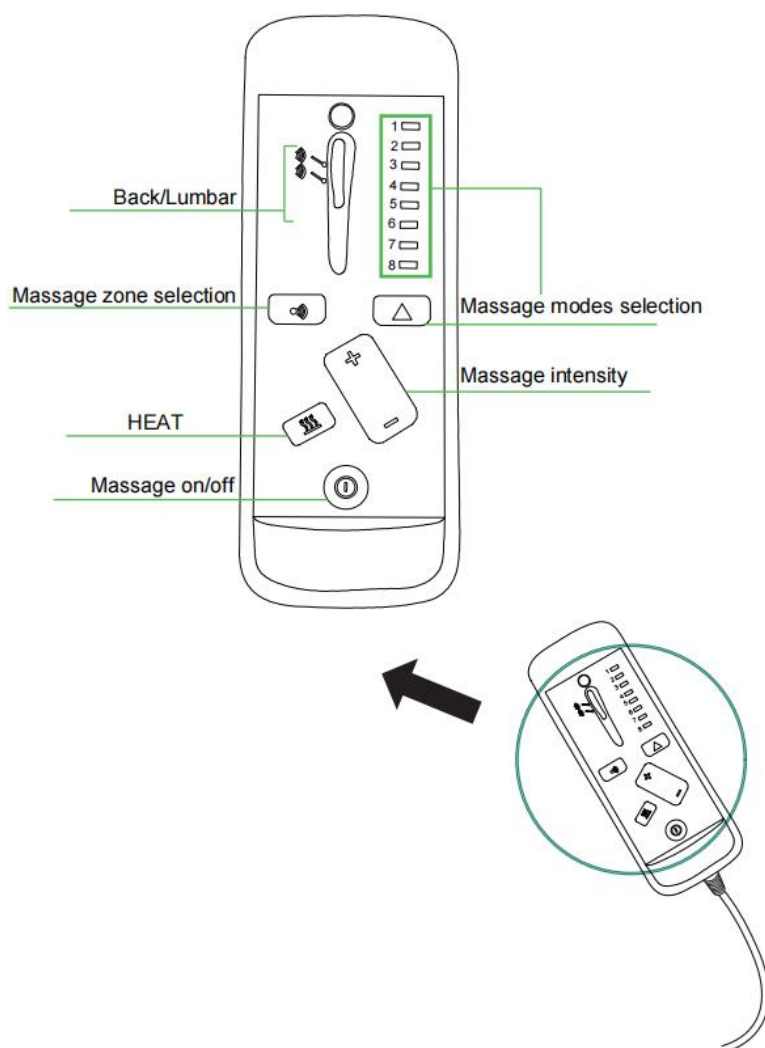
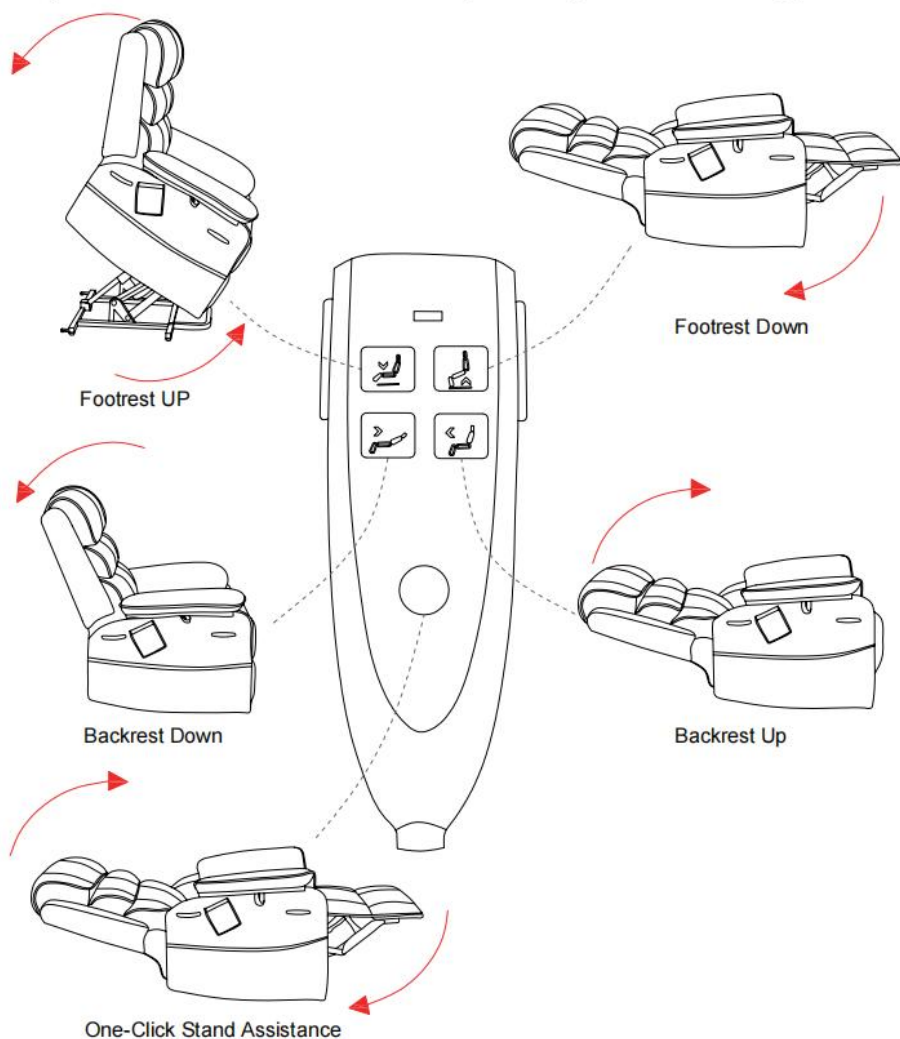
WARNING

1. To ensure the best user experience, gently pat the product after assembly to allow the foam to spread evenly.
2. After assembly, let the product sit undisturbed for 24-48 hours to allow it to fully expand and regain its shape.

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YH CONSULTING LIMITED. C/O YH Consulting
Limited Office 147, Centurion House, London
Road, Staines-upon-Thames, Surrey, TW18 4AX
H2YHUK@gmail.com

Tel.: +44 07514-677868

Upgrade · The Home Creator Way

LIFT RECLINER CHAIR

**MODEL: SX-81726-US-G/SX-81726-US-B
SX-81726-EU-G/SX-81726-EU-B**

**MODEL: SX-81726-US-G/SX-81726-US-B
SX-81726-EU-G/SX-81726-EU-B**

Lift Recliner Chair Assembly Instruction Video:

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

	Warning-To reduce the risk of injury, user must read instructions manual carefully.
	Compliance is a CE security certification.
	CORRECT DISPOSAL This product is subject to the provision of European Directive 2012/19/EC. The symbol showing a wheeled bin crossed through indicates that the product requires separate refuse collection in the European Union. This applies to the product and all accessories marked with this symbol. Products marked as such may not be discarded with normal domestic waste, but must be taken to a collection point for recycling electrical and electronic devices.
	Warning- For Indoor Use Only
	FCC Information: CAUTION: Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment! This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: 1) This product may cause harmful interference. 2) This product must accept any interference received, including interference that may cause undesired operation. WARNING: Changes or modifications to this product not expressly approved by the party responsible for compliance could void the user's authority to operate the product. Note: This product has been tested and found to comply with the limits for a Class B digital device pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This product generates, uses and can radiate radio frequency energy, and if not installed and used in accordance with the

	instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this product does cause harmful interference to radio or television reception, which can be determined by turning the product off and on, the user is encouraged to try to correct the interference by one or more of the following measures.
--	---

- Reorient or relocate the receiving antenna.
- Increase the distance between the product and receiver.
- Connect the product to an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for assistance.

SAFETY INSTRUCTIONS

WARNING:

Read this material before using this product. Failure to do so may result in serious injury.

Assembly precautions

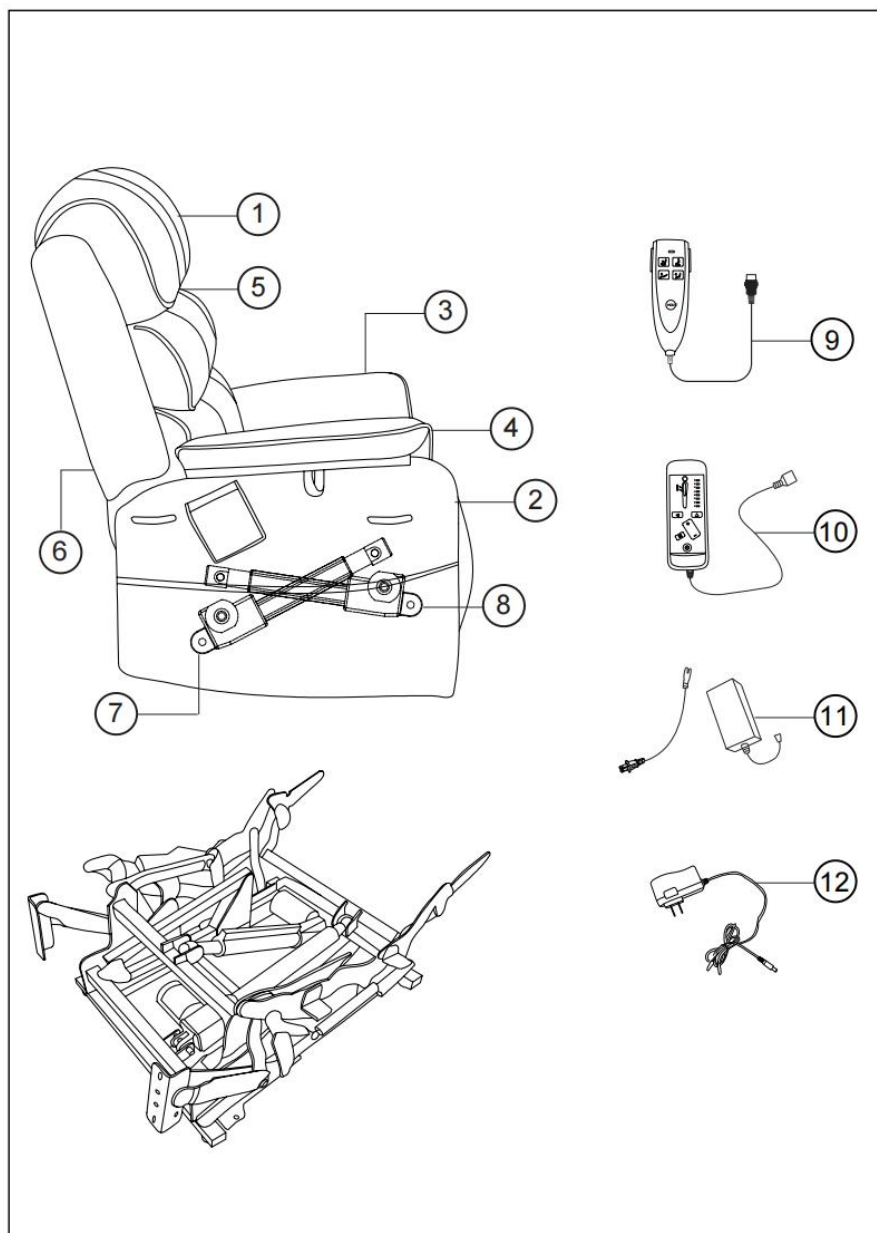
49. Assemble only according to these instructions. Improper assembly may create hazards.
50. Wear ANSI-approved safety goggles and heavy-duty work gloves during assembly.
51. Keep the assembly area clean and well-lit.
52. Keep bystanders out of the area during assembly.
53. Do not assemble if tired or when under the influence of alcohol, drugs or medication.
54. The product capabilities apply to properly and completely assembled products only.
55. Assemble on a flat, level, hard and smooth surface capable of safely supporting the Lift Recliner Chair.
56. For additional information regarding the parts listed in the following pages, please refer to the Assembly Diagram of this manual. Unwrap and separate all parts in a clean work area.

Use precautions

19. Use as intended only.
20. Do not exceed weight capacities. Empty Lift Recliner Chair before moving.
21. Inspect before every use; do not use if parts are loose or damaged.

SAVE THIS MANUAL


PRODUCT STRUCTURE DIAGRAM


NO.	Name	NO.	Name
1	Backrest	7	Base motor
2	Seat	8	Back motor
3	Left armrest	9	Motor controller
4	Right armrest	10	Massage controller
5	Left dorsal ear	11	Motor power supply
6	Right dorsal ear	12	Massage power supply

PARTS LIST

Item no.	Reference Image	Qty.
A		X1
C		X1
E		X1
G		X1
I		X1
K		X1
M		X1
O		X4
Q		X4

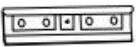Item no.	Reference Image	Qty.
B		X1
D		X1
F		X1
H		X1
J		X1
L		X2
N		X4
P		X2
R		X1


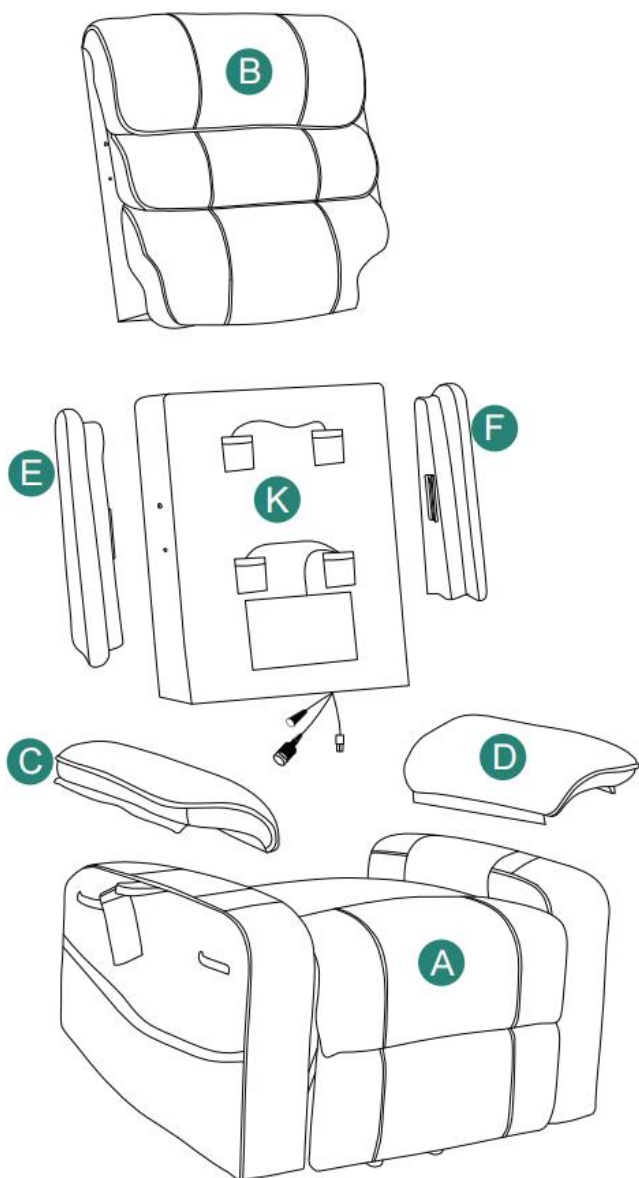
ASSEMBLY STEP


1

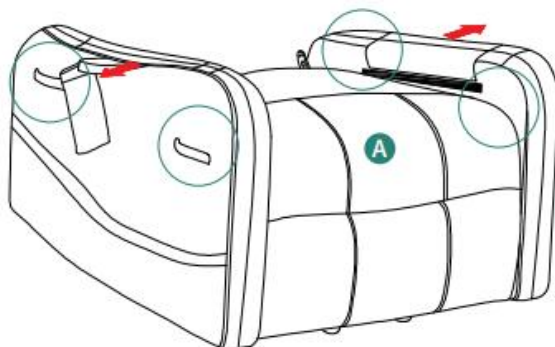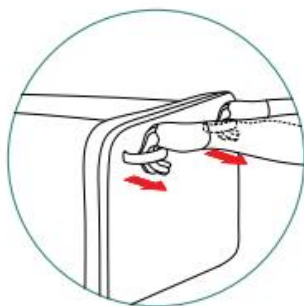
Ⓐ 1pc

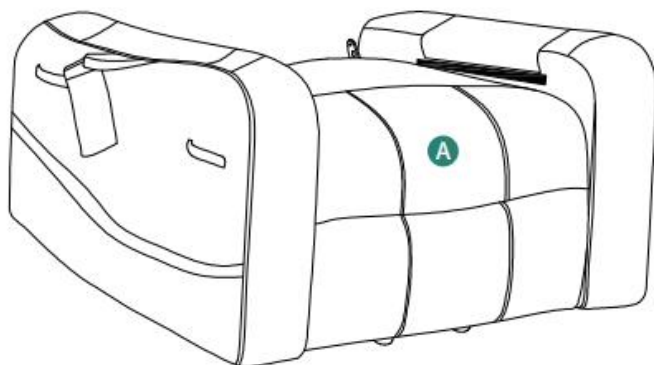
Pull arms of the chair until you hear a click.


2

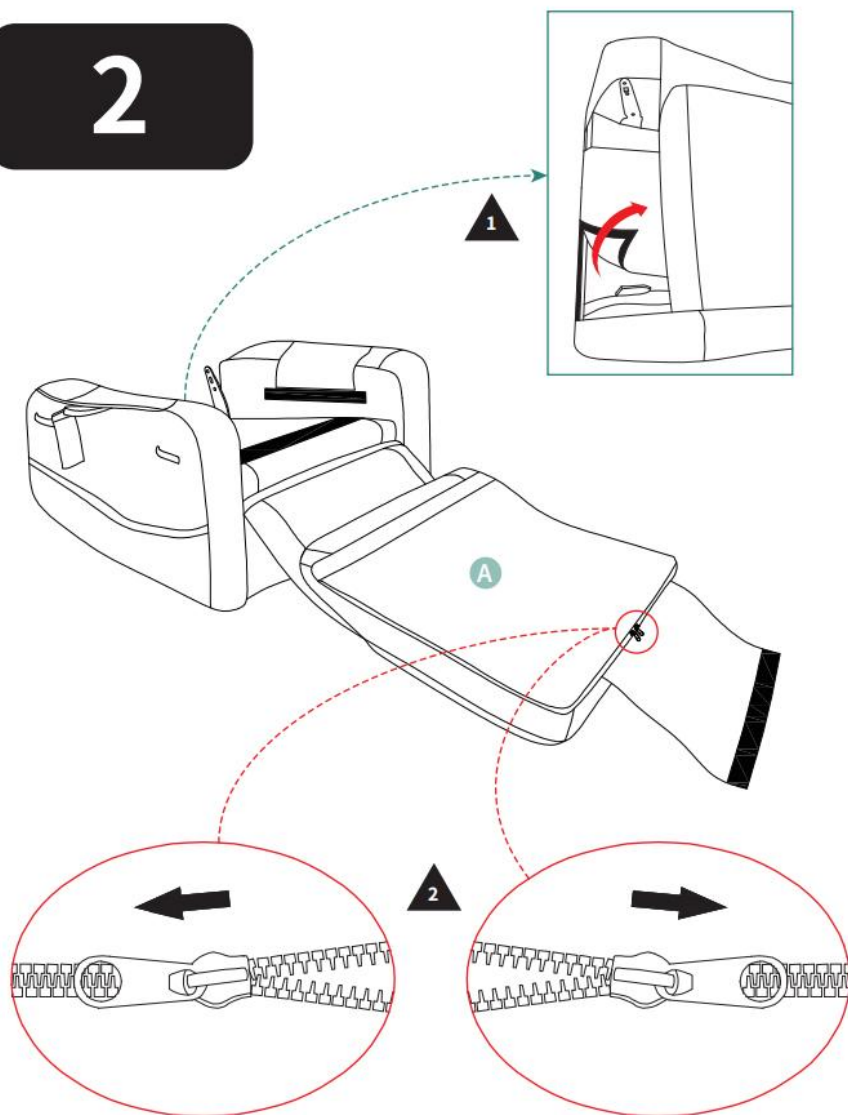
3

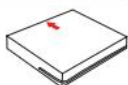Ⓔ1pc




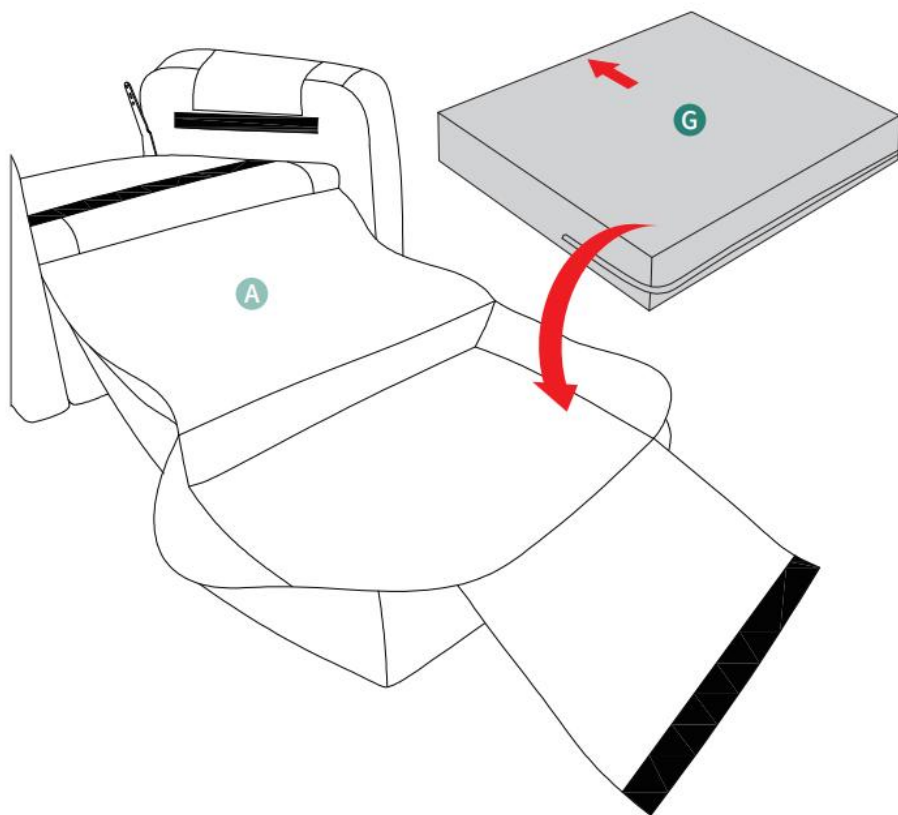
4

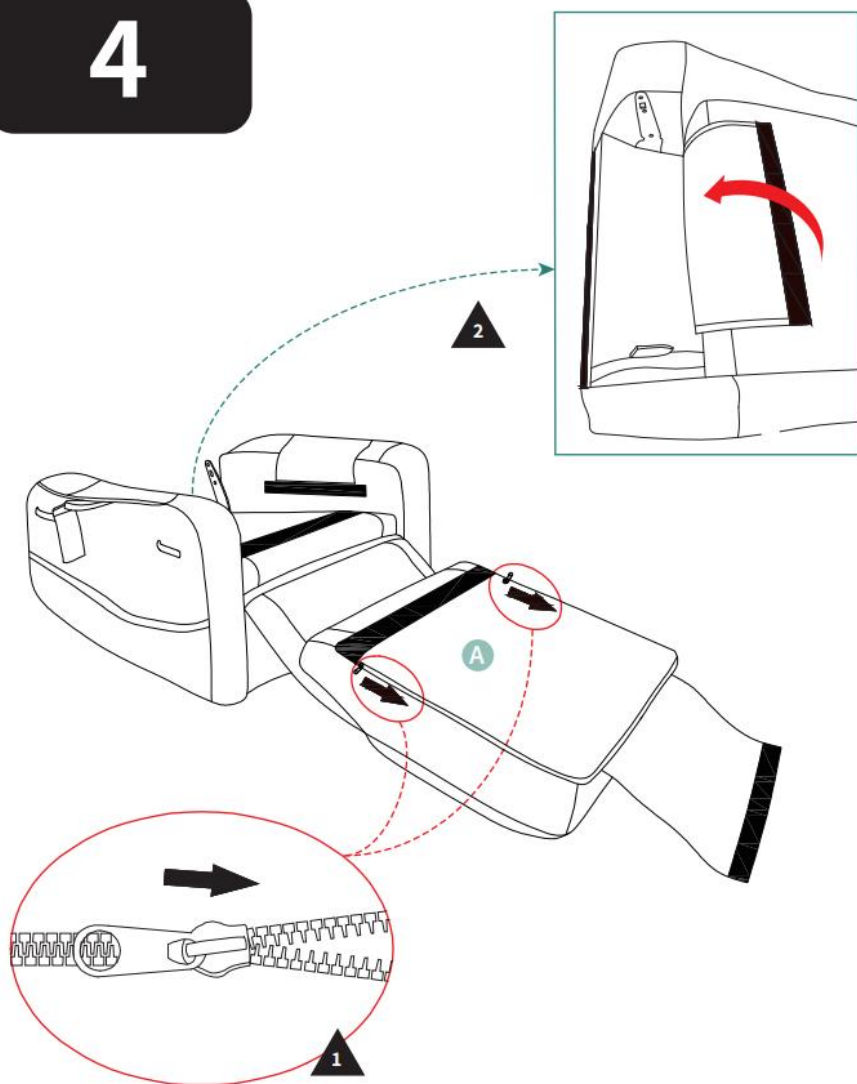5

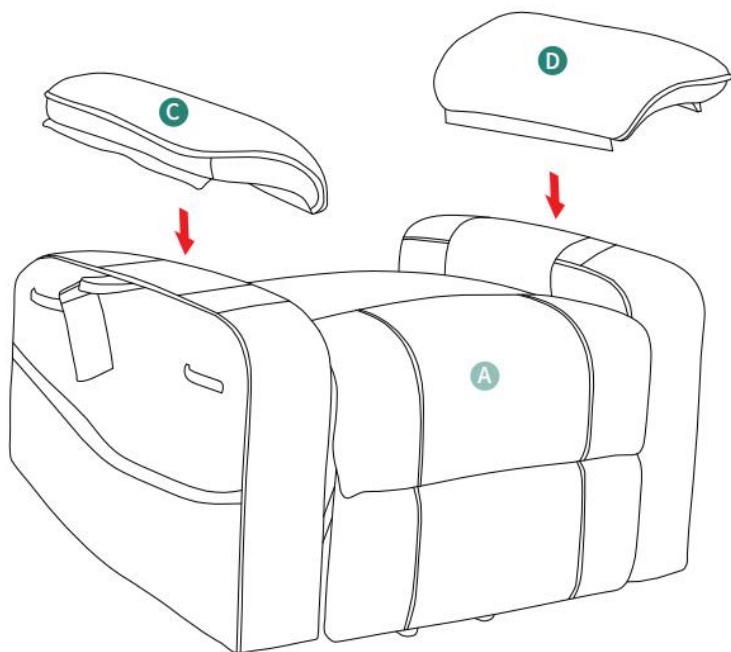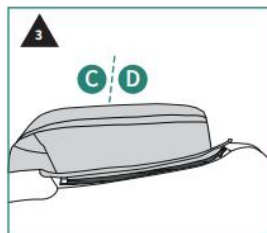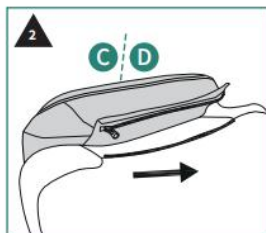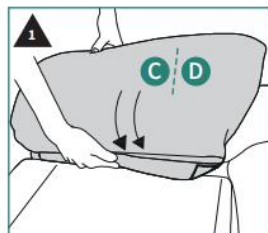©1pc ④1pc






6

Ⓑ 1pc

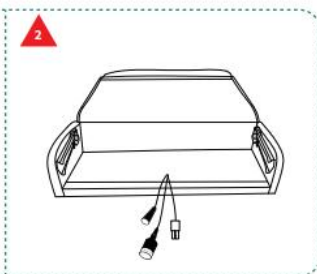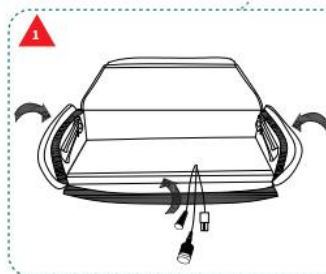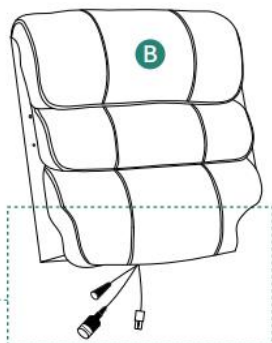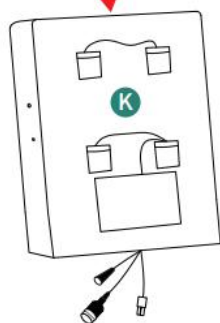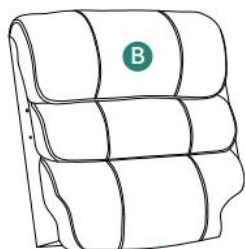Ⓚ 1pc



7


⒫ 2pc




Ⓖ 4pc

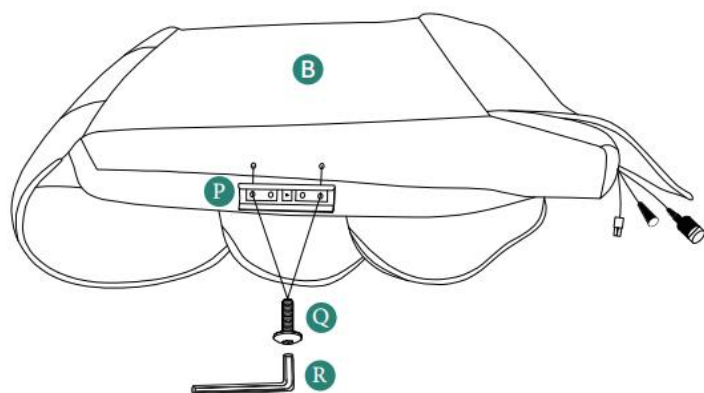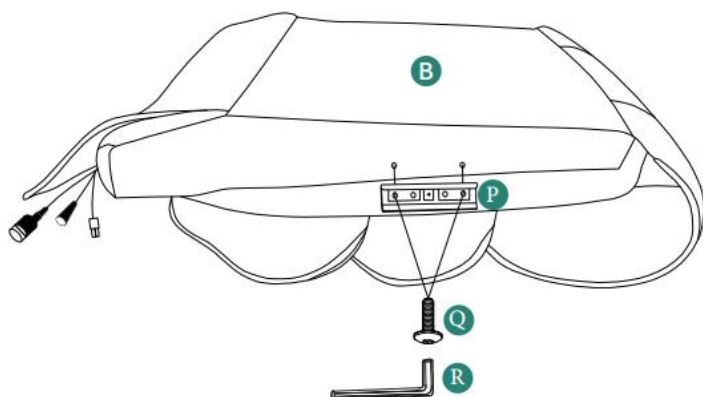Ⓑ 1pc

8

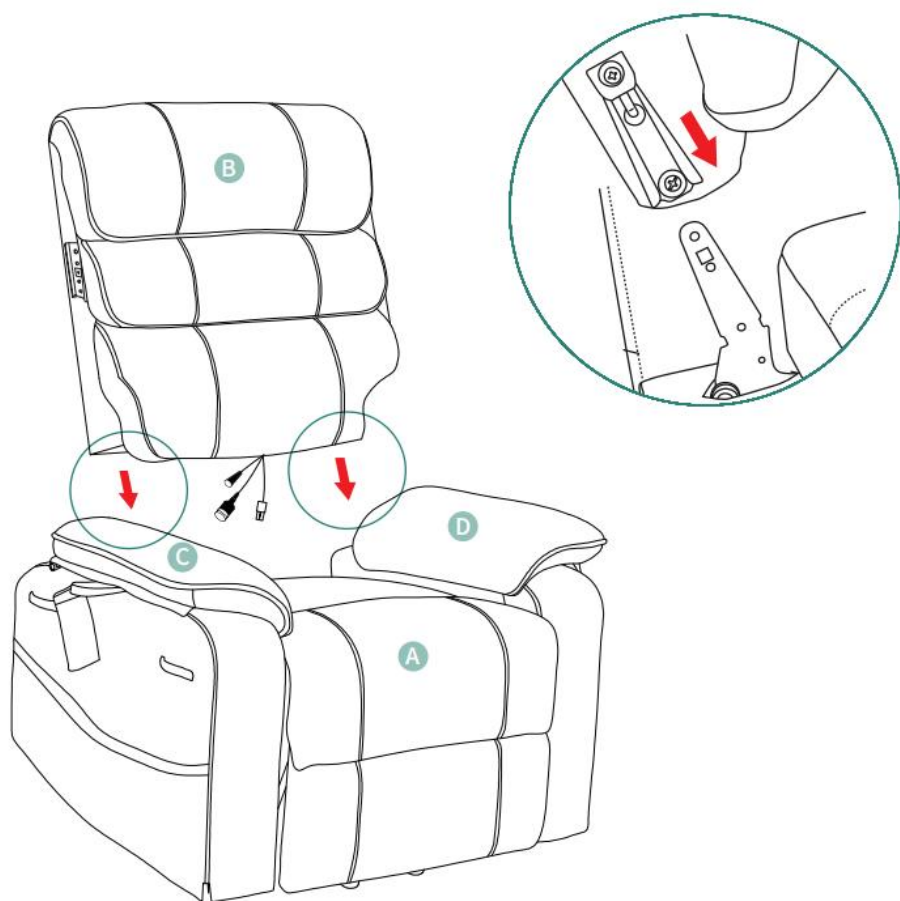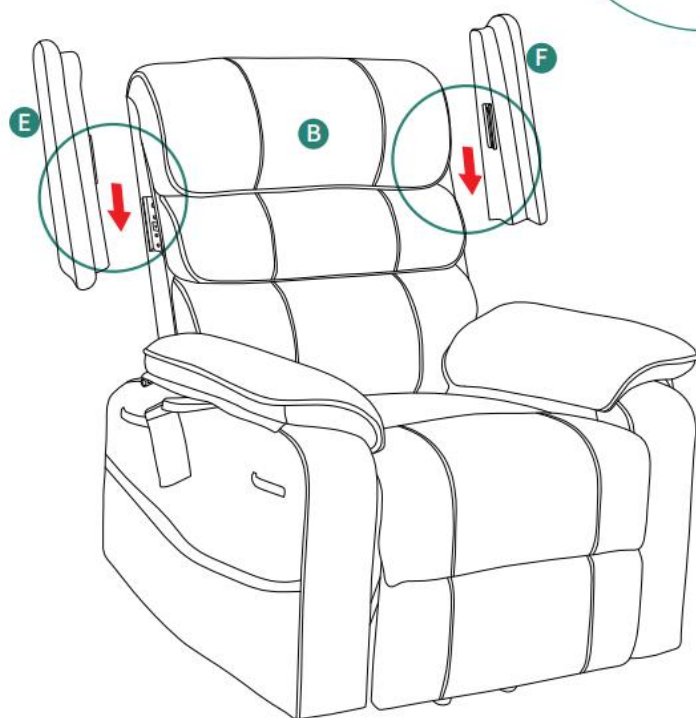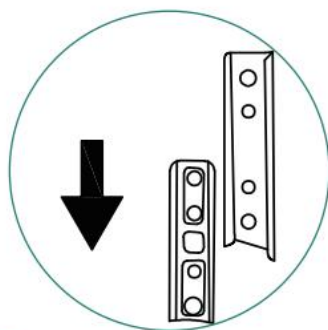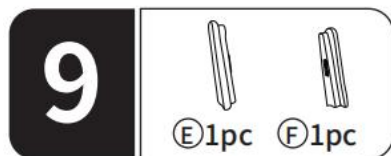10

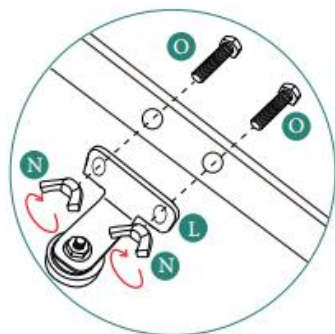
Ⓕ 2pcs

Ⓖ 4pcs

⓪ 4pcs

11

(H)1pc

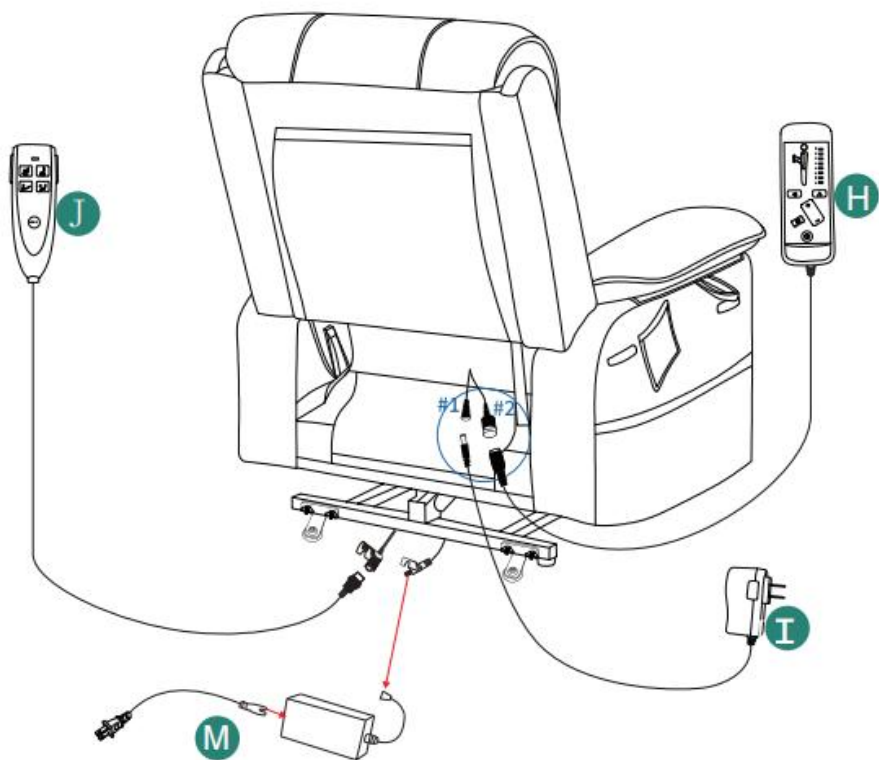
(I)1pc

(J)1pc

(M)1pc


Connect wire #1 to part I , wire #2 to part H.

12


Installation Completed

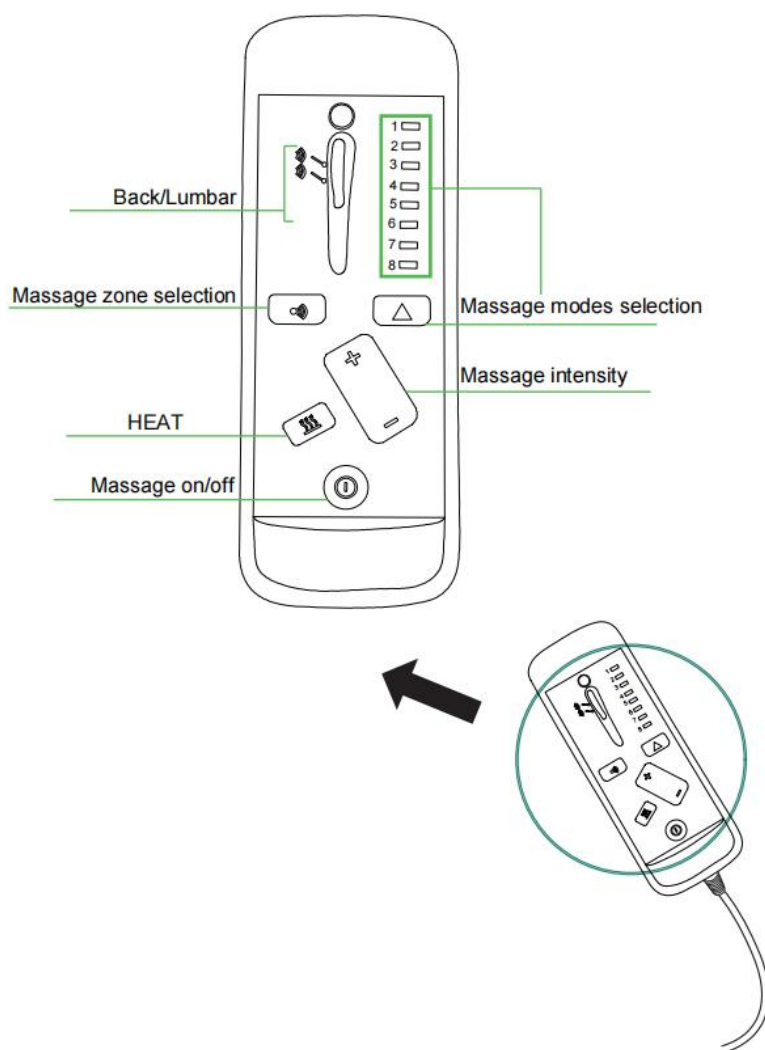How to use the massage function?

INFINITE POSITION

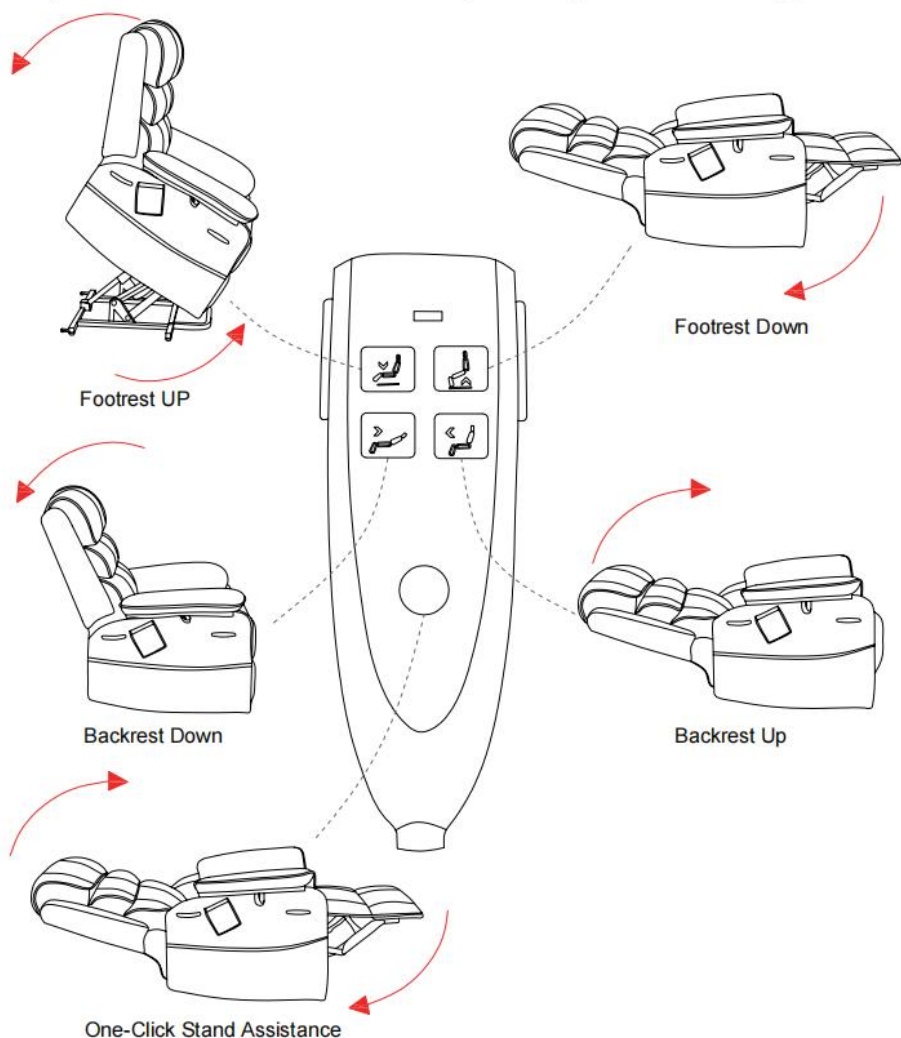Adjust the backrest and footrest independently to a full reclining position

PRODUCT PARAMETER

Model	SX-81726-US-G	SX-81726-EU-G	SX-81726-US-B	SX-81726-EU-B
Main Color	Dark gray		Brown	
Motor Electrical Rating(s)	DC29V 1.8A (Via 100-240V 50/60Hz 2.5A adapter)			
Massager Rating(s)	DC12V 1A (Via 100-240V 50/60Hz 0.5A adapter)			
Single-seat load-bearing capacity	350 lbs			
Product Size(D*W*H)	900*940*1080mm			
Net Weight	53 kg			

* Products such as specifications, appearance, and design are subject to modification without prior notice.

WARNING

13. To ensure the best user experience, gently pat the product after assembly to allow the foam to spread evenly.
14. After assembly, let the product sit undisturbed for 24-48 hours to allow it to fully expand and regain its shape.

Manufacturer: Shanghaimuxinmuyeyouxiangongsi

Address: Shuangchenglu 803nong11hao1602A-1609shi, baoshanqu,
shanghai 200000 P.R. China E-mail: support@vevor.com

Imported to AUS: SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD
NSW 2122 Australia

EC	REP
-----------	------------

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Mainzer Landstr.69,60329 Frankfurt am Main.

e-crossstu@outlook.com

Tel: +49 69332967674

UK	REP
-----------	------------

YH CONSULTING LIMITED. C/O YH Consulting
Limited Office 147, Centurion House, London
Road, Staines-upon-Thames, Surrey, TW18 4AX
H2YHUK@gmail.com

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Importerad till Australien: SIHAO PTY LTD. 1 ROKEVA STREET
E-CrossStu GmbH