EN

Before You Start

- ⚠ Please read all instructions carefully.
- ⚠ Retain instructions for future reference.
- ⚠ Separate and count all parts and hardware.
- ⚠ Read through each step carefully and follow the proper order.
- ⚠ We recommend that, where possible, all items are assembled near to the area in which they will be placed in use, to avoid moving the product unnecessarily once assembled.
- ⚠ Always place the product on a flat, steady and stable surface.
- ⚠ Keep all small parts and packaging materials for this product away from babies and children as they potentially pose a serious choking hazard.

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The design and manufacture are both conform to related product standards.

This product can be used for home health care and daily life. It does great favour in helping user standing up and sitting down. Please ensure these instructions are fully read and implemented. Failure to do so may result in injury to the user. Retain in a safe place for future reference.

Warning

- Please ensure fully understand the warning as below before use the products.
- Please ask agency or health specialist in case of any damage to body and rail.
- Check fixed parts correctly and firmly fixed.
- Any abrasion or invalid of the rail, please ask a professional to maintain or change.
- Uneven and wet, slippery ground is not available.
- Clean regularly, wiping down with a soft cloth and a mild detergent/disinfectant solution.
- Do not use corrosive solution and abrasive cleaners.

Main Feature

Max loading: 136kg

Main material: Carbon steel

Unpacking

- This product is supplied flat packed and requires assembly before use.
- No tools are required.
- Check the product from its packaging in case of any damage. If there is any fault, do not assemble the product, contact us immediately.

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1

Assemble lower foot tubes into main toilet frame. Make sure all the marbles in the same level. Rotate two handgrips through 90 degrees till marbles pop up in two sides.

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2

Pull two handgrips to opposite direction. When two holes in inner and outside tube coincide with each other, insert C-clips in two sides.

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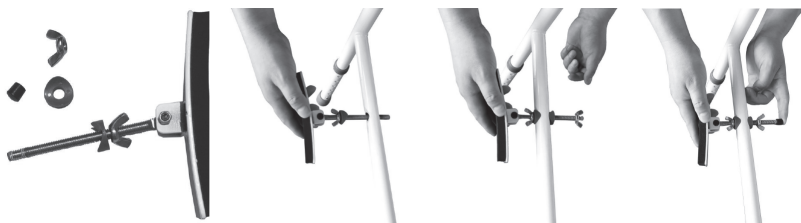
3

Detach the small plug, nut and arc sheet from the adjustment shaft, then insert the shaft into the hole of handrail tube from outside. Put all the parts on the shaft again. Do the same in the other side.

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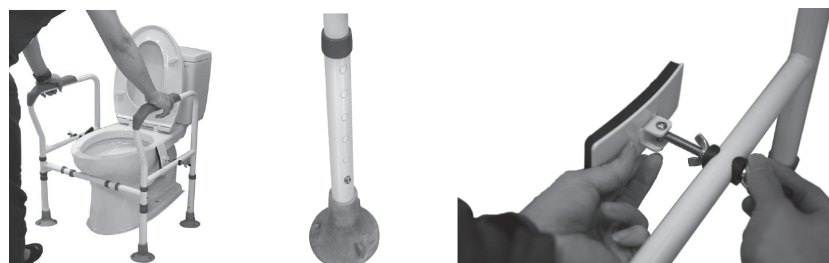
4

Lay toilet rail around the commode, and adjust marbles of lower foot tube in the suitable height for user. Adjust marbles in horizontal bar for suitable width. Then fasten commode bowl by screwing up butterfly nuts of clamps.

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5

Cross black safety bandage from back of toilet rail, then buckle the plastic fastener.

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NOTE:

Pull the ear of suction pad up before move the toilet rail.

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