Affordable. Reliable. Home Improvement.

PET WHEELCHAIR

MODEL: SS1-01 SS1-02 SS1-03

VEVOR

Affordable. Reliable. Home Improvement.

PET WHEELCHAIR

SS1-01 SS1-02 SS1-03

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

SPECIFICATIONS

Model	Size
SS1-01	XS
SS1-02	S
SS1-03	M

Pet Wheelchair User Guide

According to the after-sales feedback from users, most pets take about 1 month to fully get familiar with, adapt to and use wheelchairs.

How to Help Your Pet Get the Most Out of Their Wheelchair

Environmental Adaptation

After you have assembled the wheelchair, bring it in your pet’s favorite activity area. Leave the wheelchair there and let your pet check it out and get comfortable being around it (i.e. sniffing, touching).

Keep the area quiet and and don’t let other pets or distractions come into the area.

Harness Training

Let your pet get used to the front/rear leg straps without putting the straps on the wheelchair. Let them try the front and rear separately at first, once they are ok with wearing the straps separately try putting them in the whole harness without the wheelchair. Remember to offer treats or other rewards whenever they show comfort and familiarity to the straps

Conduct 3-5 short wearing sessions daily, each not exceeding 10 minutes. Connect the wheelchair frame only after your pet shows no resistance to wearing the whole harness.

Phased Training Steps

Stage 1: Building Positive Associations

Immediately reward your pet with high-value treats after placing them in the wheelchair.

Guide your pet to move forward 1-2 steps using handheld treats, rewarding upon completion.

Train 3 times daily, 5 minutes per session.

Stage 2: Short-Distance Movement

Use a leash to assist with directional control.

Practice straight-line walking in an open, obstacle-free area.

Reward every 3 meters of successful movement, gradually increasing the distance between rewards

Stage 3: Complex Environment Adaptation

Introduce slight turns and furniture avoidance training.

Provide manual assistance for the first time passing through doorways or obstacles.

Offer more rewards after completing complex maneuvers.

Key Precautions

Safety Control

Use a short leash (≤ 1 meter) for the first 3 days.

Avoid hazardous areas such as stairs or slopes.

Immediately disconnect the harness and comfort your pet if they show signs of struggle or panic.

Usage Duration Management

Week 1: Single sessions should be no more than 15 minutes, daily total no more than 1 hour.

Week 2: Gradually extend to 30 minutes per session, daily total no more than 2 hours.

Long-term use: Check for limb compression once for every hour of use.


Behavioral Observations

Support your pet's abdomen during urination/defecation.

Check strap tightness if frequent chewing or licking occurs.

If your dog refuses to move, you can try changing the type of treat you are offering.


Components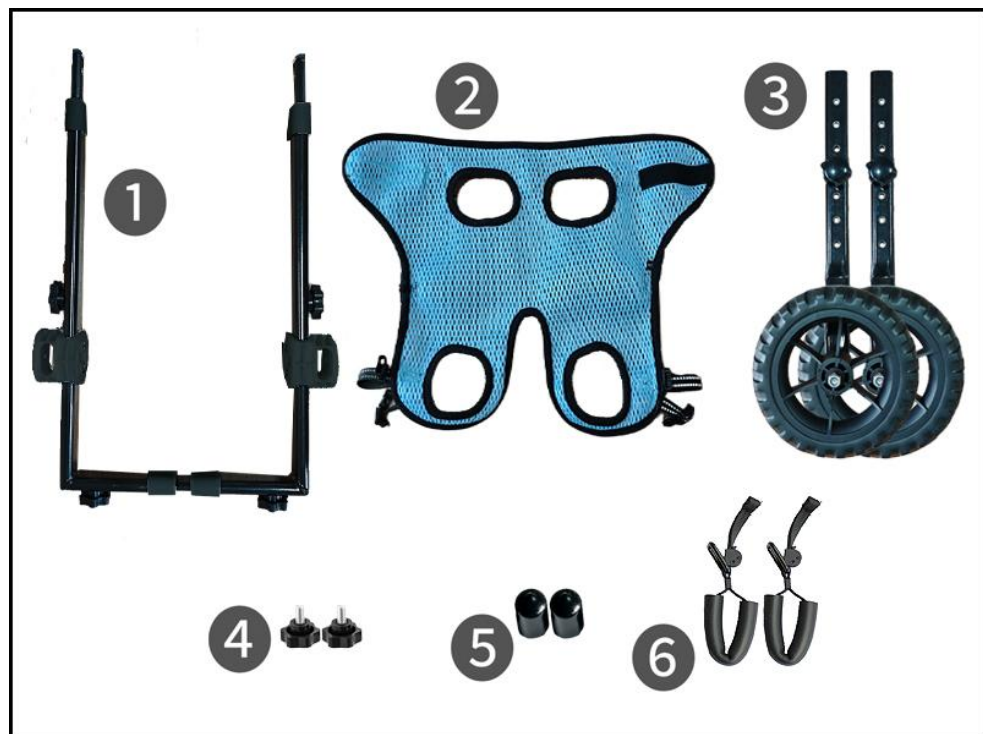


No.	Parts	Qty.
1	Support Frame	x1
2	Integrated harness	x1
3	Wheel Rod & Wheel	x2
4	M6 screws	X2
5	Soft rubber cover	X2
6	Sponge Circle	X2

Installation and length adjustment instructions for the frame


Step 1: Measure the dog's body length as shown in the figure, locate part ❶, the metal frame. Unscrew the screws and pull out the inner tube to match the length of your dog. Pick the hole closest to your measurement and put the screw back in.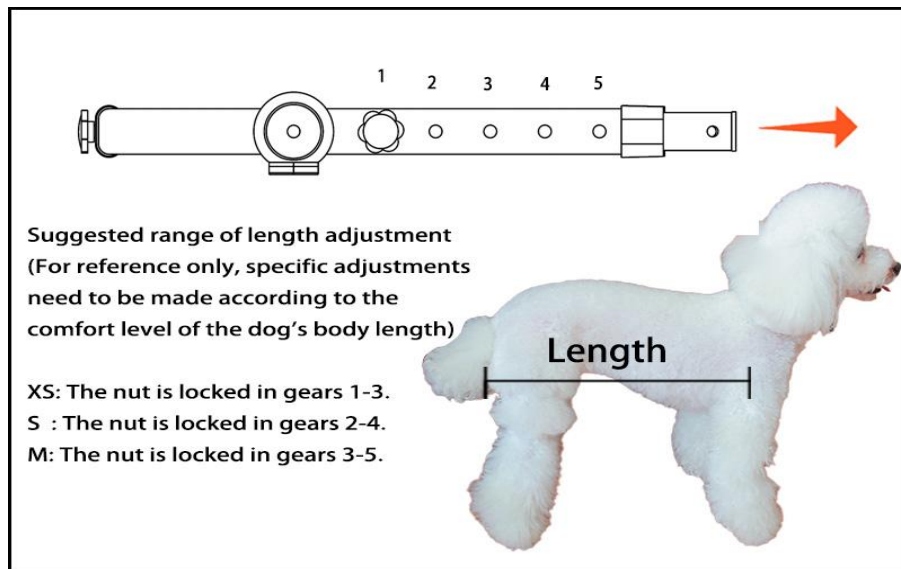

Step 2: Measure the width of the dog's buttocks, unscrew the screw and adjust the frame width until it matches your measurements. pick the hole closest to your measurement and then put the screw back in.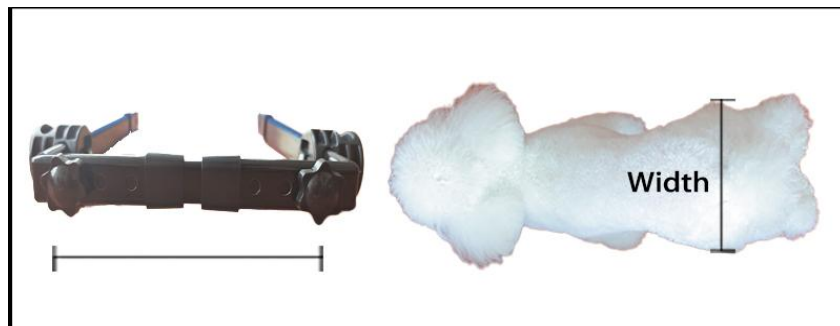

Step 3: Measure the height of the dog (the vertical distance from the hind limb joints to the foot pads). Find part ❶ and part ❸. Unscrew the screw on part ❸, insert part ❸ into part ❶, (see pics below for how it should look) match your length measurement to the closest hole on the frame and then tighten the screw to lock the wheels in.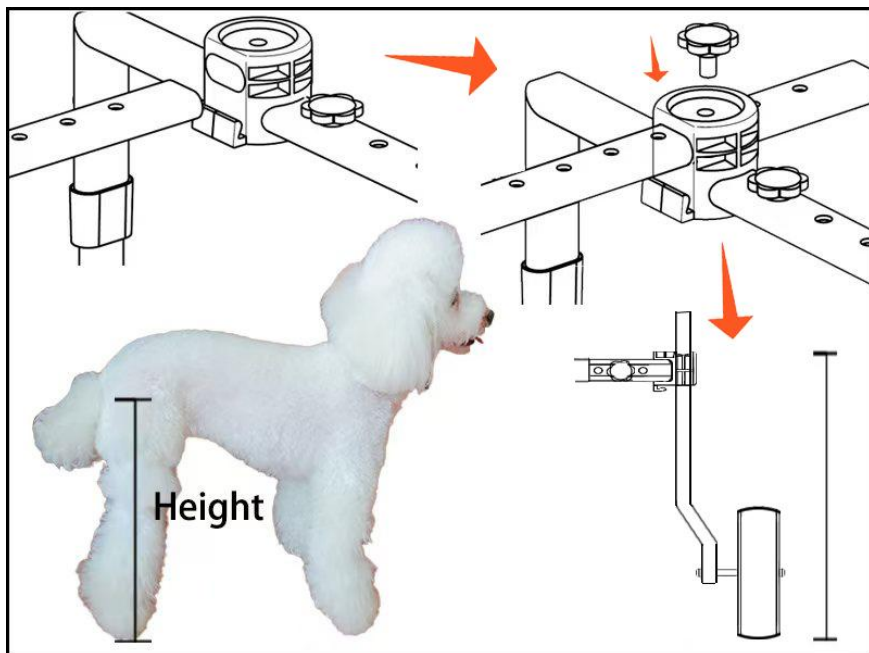

How to adjust and put on the harness


Step 1: Find part ❷ and help your dog put on the harness as seen in the picture below. Start with the front legs.


Step 2: After your dog's legs are in the harness, stick the Velcro strap together, adjust the length of the strap and lock the buckle. You can tighten the strap again after the buckle is locked, but remember to not make it too tight. You should be able to slide a finger in between the harness your dog.

Step 3: As shown in the picture, place the dog into the frame. Install part 5 onto the end of the height adjustment rod.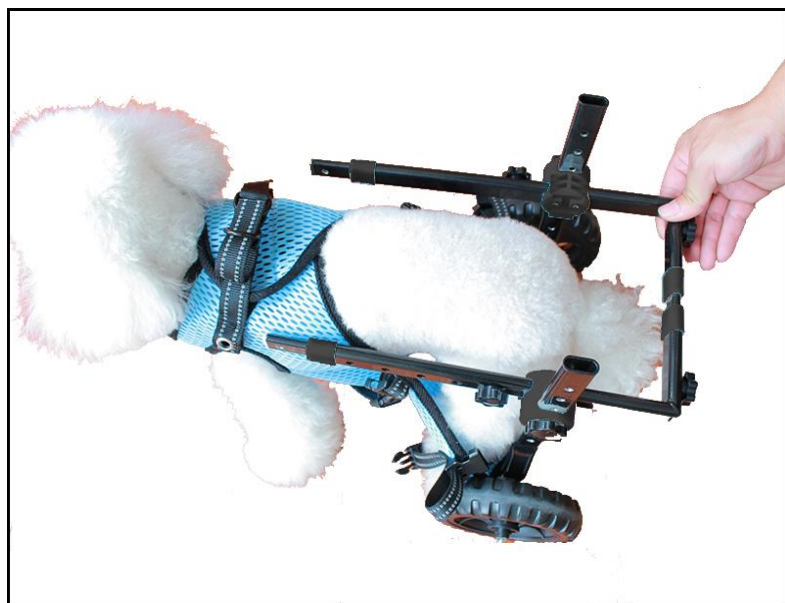


Step 4: Now find the loop with the metal holes(rivets) on the front sides of the harness. Slide the front part of the wheelchair frame through the loop. Line up the holes and use part ④, a screw, to attach the harness to the wheelchair frame. See the pictures below for more details. Do the same for both sides.

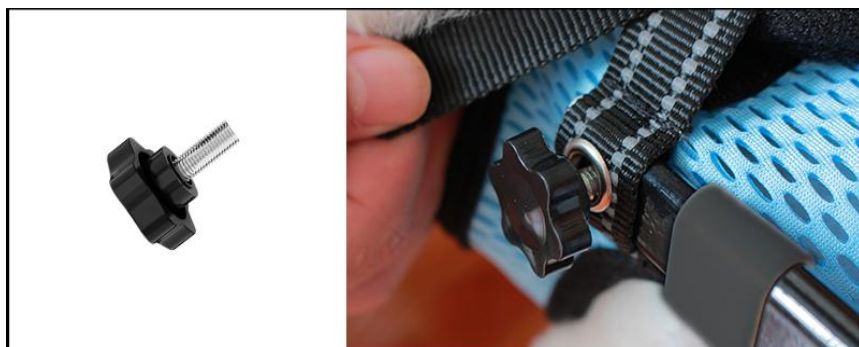Step 5: Wrap the middle straps of the harness around the wheelchair frame and press the buckle together. (Need to tighten the strap)


Step 6: Take the rear harness straps and wrap them around plastic wheel joint as seen in the pictures below. Make sure you pay attention to the arrows in the picture for correct installation. (Need to tighten the strap)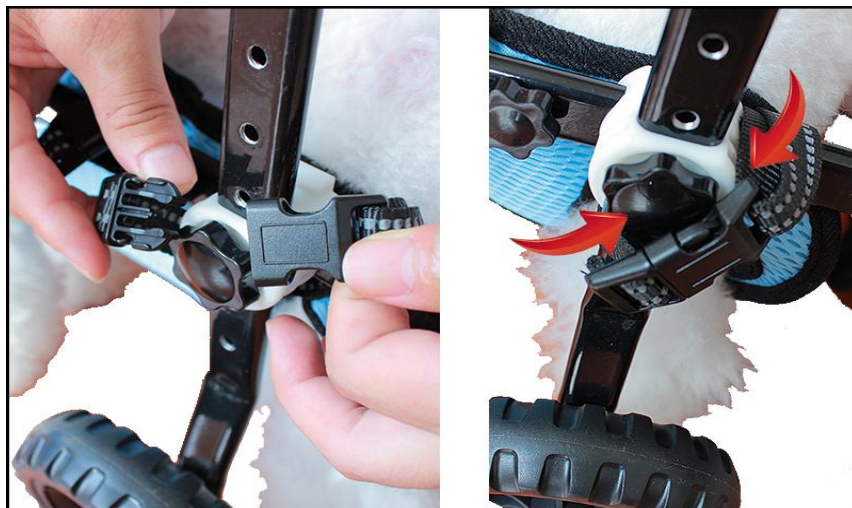


If your dog requires rear leg support, please continue to follow steps 7 and 8.

Step 7: Measure your dog's hind leg circumference. Open the buckle on part 6, slide to adjust, then press to lock until you hear a "click". Make sure the loop isn't too tight or uncomfortable for your dog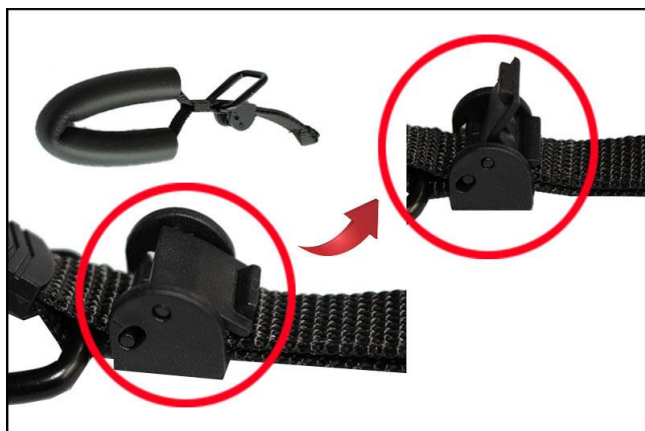

Step 8: Unscrew the screw on the width adjustment rod (back of the wheelchair) and line up the metal ring from the leg support loop with the hole where the screw was. Screw the two pieces together. **Place your dog's hind legs into the sponge loops after the loops are attached to the frame**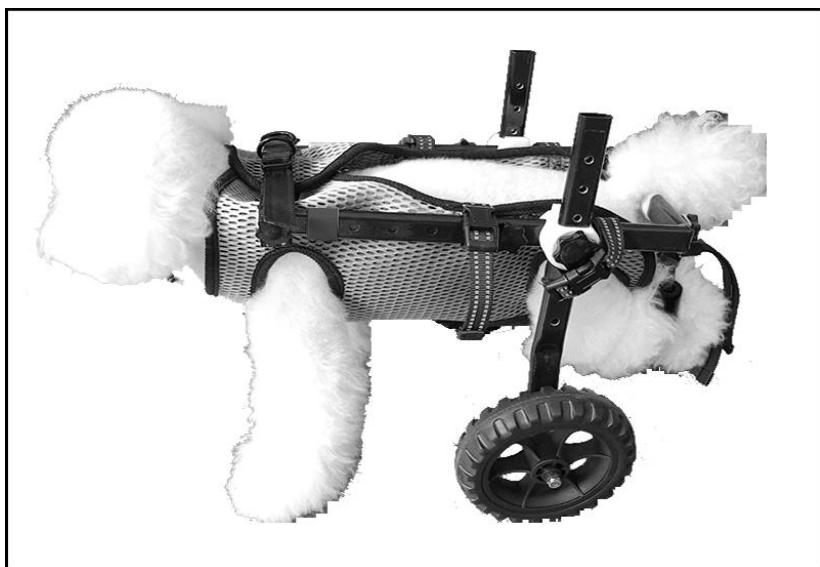

Maintenance and adjustment tips

Daily Checks

Inspect screw tightness and bearing flexibility weekly.

Make sure to hand tighten all screws before use.

Dry the frame after rainy days to prevent rust.

Clean the harness regularly with a pH-neutral detergent.

Size Adjustment

Chest circumference: Allow a one-finger gap for comfort.

Frame height: Ensure leg joints remain naturally bent when standing.

Re-measure your dog every 3 months based on growth/weight changes and make necessary size changes when needed.

FAQs & Troubleshooting

Tips on how to help your dog get used to using its wheelchair

Q: My pet refuses to move or shows anxiety. Is the wheelchair not working?

This is normal during the first few times it tries the wheelchair. Stay patient.

Revisit the "Phased Training Steps" for help on how to encourage your dog to use the wheelchair.

Use higher-value treats (e.g., fresh meat) during training if normal treats don't seem to help

Avoid forcing usage; keep daily training within the recommended durations.

Q: My pet is moving backwards, not forwards

Check the Chest Strap First! The most common issue causing this issue is the chest strap.

. Make sure it sits behind the sternum (breastbone) and before the ribs—never pressing the armpits (causes pain) or belly (prevents pushing). All straps should be snug but not tight, remember that you should be able to easily slide a finger between the strap and your dog.

Q: Why aren't my wheels turning smoothly?

Check the wheels to make sure there is no hair or foreign objects stuck in them

Q: Do I have the correct height?

Your dog's paws must be touching the ground to push naturally. Remember that their joints should have a natural bend to them.

Q: How can I keep my dog from slipping on the floor when learning to use the wheelchair?

Switch locations immediately! Tile/wood floors can cause slipping. Only practice on carpet, grass, or rough ground.

Q: How can I help address fear/resistance to wheelchair use:

Remember to stick to the short acclimation sessions: Start with getting used to the harness

first, then try the harness and wheelchair together. Put the wheelchair on multiple times daily—no movement required. Keep the focus on getting comfortable being in it. Reward with treats/praise to build positive associations, even just for small successes.

Q: How can I help my dog learn to walk with the wheelchair?

Guide with treats and always be gentle with them through the whole process.:

Lightly support the belly/hips while slowly nudging them forward.

Lure them forward with a favorite treat/toy held close to the front of their nose.

Instantly reward any forward attempt with praise + treats!

Key Notes:

Never force or punish backward movement! This can worsen fear.

Patience is vital! Progress takes time—even standing calmly is a win.

Q: Why does a dog refuse to move while in a wheelchair, or lean to the side after taking just a few steps?

It may be because dogs have highly sensitive hearing, and the sound produced by the wheelchair wheels touching the ground during movement is an unfamiliar noise that scares them. In such cases, owners need to comfort their pets and must not force them to walk by pulling. Instead, they can demonstrate the sliding of the wheelchair frame and wheels in front of the pet multiple times to help it get used to the sound.

Sizing Issues

Q: How can I ensure proper wheelchair sizing?

How to measure for length: Measure from the end of the sternum(breastbone) to the base of the tail.

Width adjustment: Measure the widest rib span plus 2cm allowance.

Height adjustment:

Align the frame with the highest point of your dog's hips.

CAUTION:Avoid positions that are too high (can cause abdominal pressure) or too low (hind legs dragging).

Q: What should I do if the size is incorrect?

When you receive the wheelchair, please note that we've set the length extension rod to its smallest position during packaging. You'll need to install the other width and height adjustments yourself.

Length extension rod adjustment tips: Some customers have had problems where they move the spring loaded ball from the last hole to the first hole and then stop pulling. However, in this case, the length extension rod has only been pulled out one-third of the way. You can continue pulling out the length extension rod to get the correct length for your dog.

Q: How do I adjust the harness?

Tighten the abdominal strap until one finger can slide in.

Ensure the chest strap allows free breathing while still being snug and supportive.

Check strap tightness weekly; adjust if weight changes $\pm 10\%$.

Q: What can I do if parts are lost or damaged?

Confirm the part number using the "Component Coding Diagram" in the manual.

Take clear photos of the damaged/lost part.

Contact customer service personnel through the order page.

Critical Reminders

Immediately stop use and contact support if:

Metal frame shows abnormal deformation/cracks.

Wheel bearings produce metallic friction noises.

Straps exhibit 2 or more consecutive stitching breaks.

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YH CONSULTING LIMITED.
C/O YH Consulting Limited Office 147, Centurion House,
London Road, Staines-upon-Thames, Surrey, TW18 4AX

Affordable. Reliable. Home Improvement.

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SPECIFICHE

Modello	Misurare
SS1-01	Taglia XS
SS1-02	S
SS1-03	M

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Importado a AUS: SIHAO PTY LTD. 1 ROKEVA STREET, EASTWOOD,
Importado a EE. UU.: Sanven Technology Ltd. Suite 250, 9166 Anaheim
E-CrossStu GmbH